



# Mangar Elk Quick Start Guide


See '**BATTERY NOTE**' page 2 before proceeding. Please read the full product user instructions for the Airflo 24 and ELK.

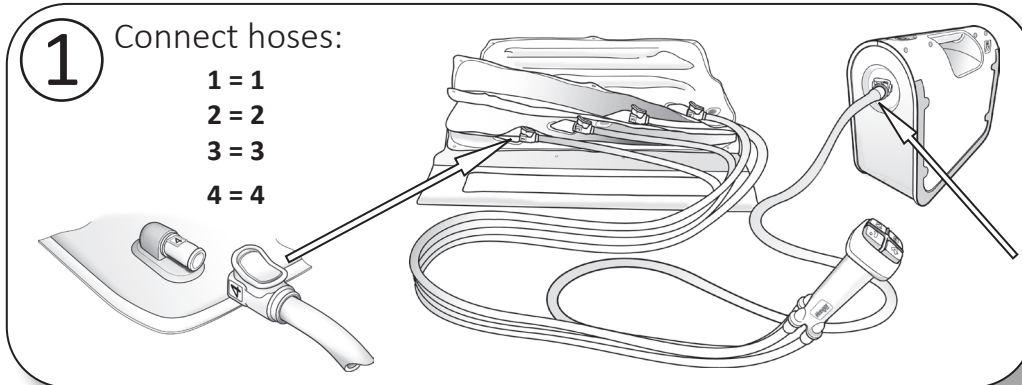


Make the Connections

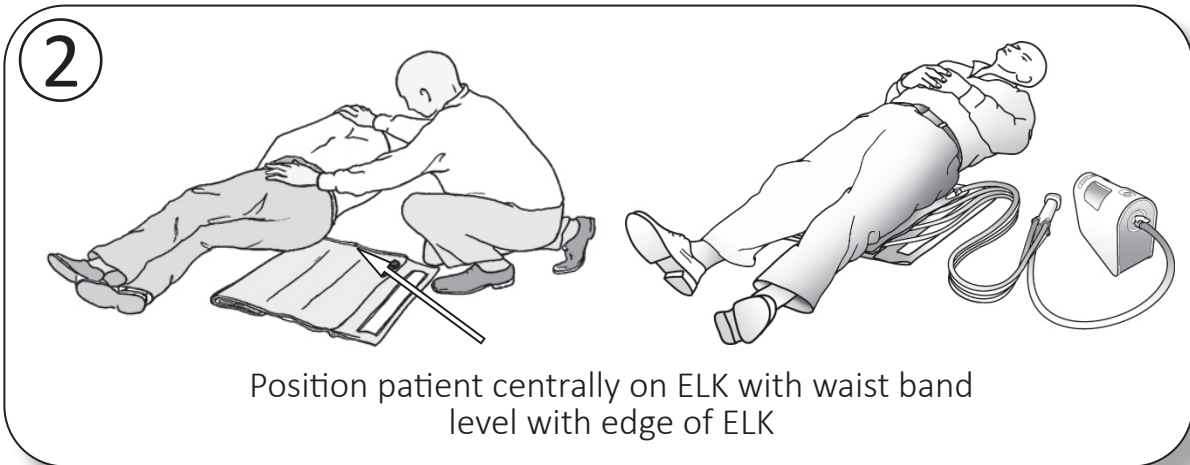


Position Patient

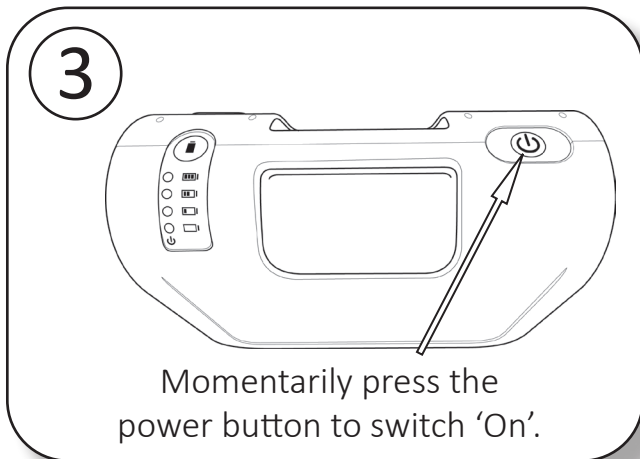

Switch 'ON'

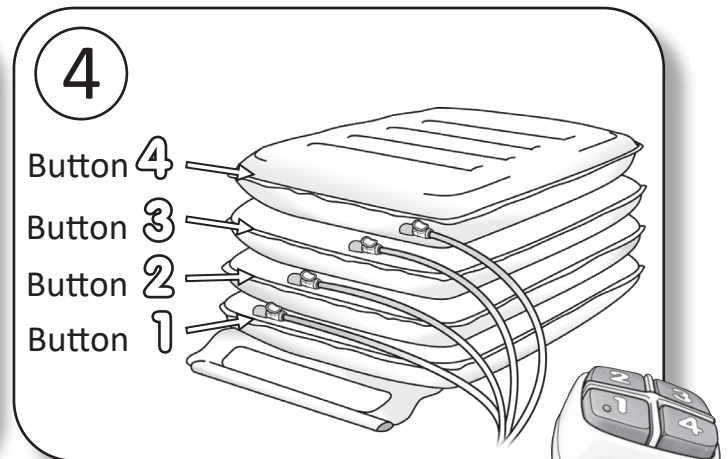


Operate Hand Control Buttons.  
(See 'Lifting Sequence' page 2)

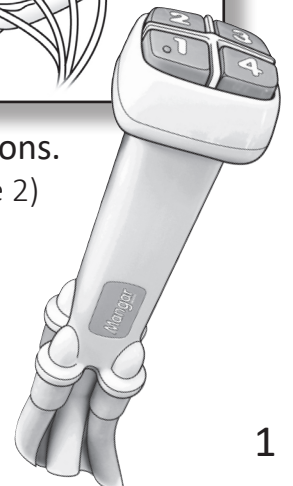


# Mangar Elk Quick Start Guide

**5** Bring the patient to a sitting position and provide support.

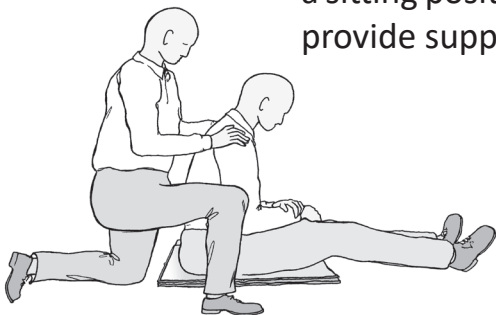

**6** Whilst providing support:

1. Fully inflate section 1
2. Fully inflate section 2
3. Fully inflate section 3
4. Fully inflate section 4



NOTE: The compressor will automatically stop as each section reaches full pressure.


**WARNING:** While in use, **never** allow a section to become partially deflated while any sections above it are inflated. Sections must always be fully inflated from the bottom up, in sequence, in order to maintain stability.


## Cleaning and Disinfecting


The ELK should be fully inflated and pre-cleaned by washing with detergent and water, thoroughly rinsed and then disinfected using a 1:10 solution of household liquid bleach and water (1 part bleach to 10 parts cold water). Leave the solution on the ELK for approximately 10 minutes before thoroughly rinsing off with clean cold water. Allow to air dry before storage. Do not dry using hot air.


**BATTERY NOTE:** Charge for 12 hours before first use. Before each use, check the battery charge state by pressing the 'Battery State Indicator' button (*please read full Airflo 24 User Instructions*).


  
Press Battery State Indicator Button

  
**RED**  
Recharge Immediately

  
**1 GREEN**  
Recharge before next use

  
**2 GREEN**  
Recharge soon

  
**3 GREEN**  
OK to Use

  
**4 GREEN**  
OK to Use